

3 Step Smile Design


Step 1: Send photos to Hallmark Dental Lab

- Sending photos to Hallmark. Take a full face photo (smiling) and a photo with lips retracted. These are the minimum – send more if you have them.
- Describe the patient's needs (longer/whiter/close diastema/shape change)
- Let us know the patient's email when you send the case (this is how we get the results to their app).



Step 2: Have your patient download "My 3Shape For Patients" on their phone.

- Have the patient download "my3shape for Patients" on their phone. Their access will not be active until after the case is designed.

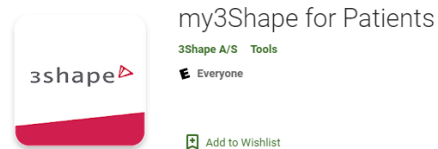

Step 3: Share The proposal with your patient. *

- When the case is ready to share, the patient will activate their app. They do this by opening the app and clicking: "**Activate Patient Account**" in the app. They will get a message that indicates "You will need to register at your dentist." We will do that for you using the patient email address you provided. They may click "Continue". You may want to do this step with the patient at your clinic – you can see their initial reaction to the design and get immediate feedback.
- After clicking "Continue" they will be asked for their email and their "temporary code" or "PIN". The PIN will be 5555.
- This will prompt the patient to create their own password. Once this is done, the patient will have access to their completed smile design photo.

* We suggest you do not share the PIN with the patient until after you (the dentist) have seen the design. One of the many values of the design is to help visualize proposed treatment. Our design might suggest treatments that you do not wish to provide. You may want to tweak the designs in other ways before having the patient see it.